Autodrop®

OP6000

Instructions

Mode d'emploi

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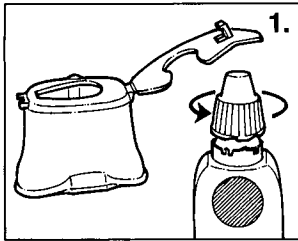
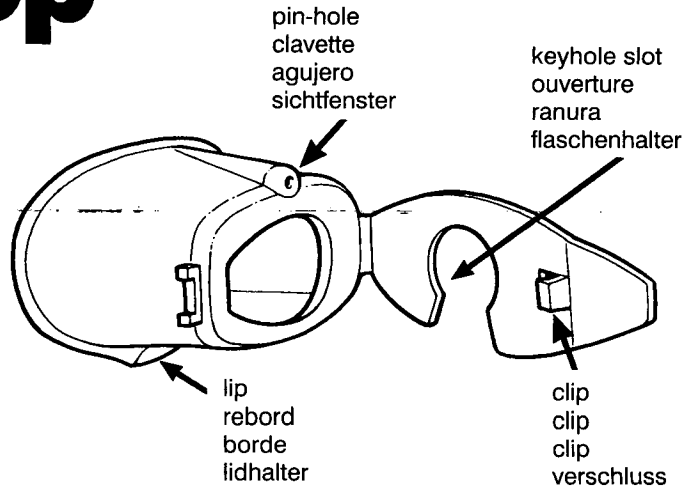
Gebrauchsinformation

Autodrop® has been developed to assist in the instillation of eye drops.

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
1. Open Autodrop. Loosen cap from eye drop bottle but do not remove completely.

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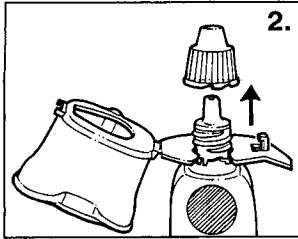
2. Locate the keyhole slot around the base of the bottle neck below the thread. Remove bottle cap.

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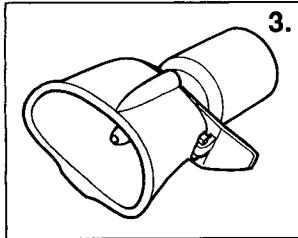3. Close Autodrop. Do not force the bottle nose into Autodrop, it should close easily.

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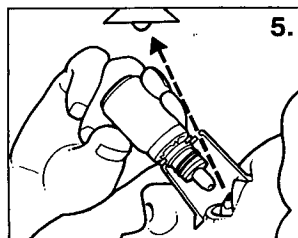
4. Hold down lower eyelid. Place Autodrop over the eye with the lip against the cheek.

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
5. Tilt the head backwards. Look through pin-hole towards the light.

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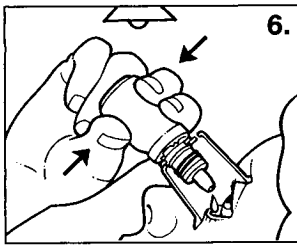
6. Squeeze bottle gently to allow prescribed drops to fall into the eye.

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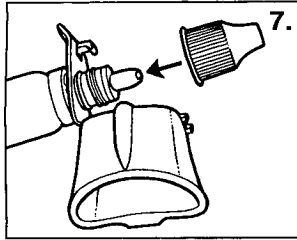
7. Open Autodrop and replace top on eye drops bottle between doses.

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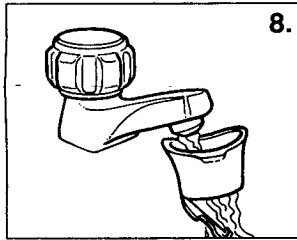8. To clean Autodrop remove eye drop bottle and wash Autodrop in warm soapy water. Rinse and dry thoroughly.

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We recommend that Autodrop is used for the whole course of treatment for an eye disorder. When an eye drop bottle is discarded remove Autodrop and clean thoroughly before fitting to the new bottle. Autodrop may be used on more than one type of eye drop at the same time but if you suffer from more than one eye complaint we recommend that you use a separate Autodrop for each type of eye drop. In hospitals and clinics: Autodrop may be autoclaved on a low pressure cycle for 30 minutes. Release clip to allow moisture to evaporate. Autodrop can also be cleaned in sterilising fluid. Rinse and dry thoroughly.

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GUARANTEE

Autodrop is guaranteed for 3 months from purchase. If Autodrop breaks within this period a full refund will be given.

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OWEN MUMFORD

www.owenmumford.com

Developed in collaboration with the RNIB

Euro patent app no 89305658 0
USA patent app no 07/480876
Worldwide patents applied for

Autodrop® is a trademark of Owen Mumford Ltd

UK
Owen Mumford Ltd
Brook Hill, Woodstock
Oxford OX20 1TU, England
Tel: 01993 812021
Fax: 01993 813466
info@owenmumford.net

NORTH AMERICA
Owen Mumford Inc
1755-A West Oak Commons Court
Marietta, Georgia 30062, USA
Tel: 800-421-6936
Fax: 770-977-2866
info@owenmumfordinc.com

FRANCE
Owen Mumford
BP 444
27204 Vernon Cedex, France
Tél: 02 32 51 88 70
Fax: 02 32 21 96 95
owenmumford@compuserve.com

GERMANY
Owen Mumford GmbH
Sternplatz 3, D-97896
Freudenberg/M, Germany
Tel: 09375/92990
Fax: 09375/929929
info@owen-mumford.de