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**DE**

### Otocura® Ohrenspray – Pflegeöl

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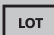
**Instructions for use**
**EN**
**Otocura® earspray - skin care oil**
**Ingredients:**

Olive oil, almond oil, calendula oil (based on peanut oil)

**Areas of application:**

For the care and hygiene of the external auditory canal, for the relief of itching due to dryness, and for the preventive and concomitant treatment of inflammation in case of skin irritation in the external auditory canal, e.g. due to improper ear cleaning or in connection with the wearing of hearing aids. Can be used for prevention and for care after various types of water sports (e.g. diving, swimming).

**Background:**

Cerumen (earwax) moistens the ear canal and preserves the acid mantle of the skin. It also protects against infections and promotes deposition of any debris such as dirt, dust and dead skin cells outward towards the outer ear. As a result, the ear cleans itself, so that usually a carefully cleaning at the end of the ear canal on the outer ear is sufficient.

Hardened earwax, excessive wax production, or malfunctioning of the self-cleaning mechanism may cause blockage of the ear canal (e.g. earwax plug). This can cause earache and reduce hearing. Otocura® can soften the cerumen to aid normal self-cleansing. Otherwise, soften the earwax blockage a few days before the doctor's visit with Otocura® so that it can ease the removal by professional methods e.g. ears syringed.

If earaches persist for a longer period and/or are accompanied by other medical conditions (e.g. fever, dizziness), consult medical service.

**Directions for use:**

If a cold fluid enters the ear, it may cause pain or dizziness. Therefore, it is recommended to warm Otocura® up to body temperature e.g. in your hand or in your pocket for a few minutes before use.

Hold the bottle vertically and place the sprayhead directly to the opening of the ear canal when applying the spray.

- For care and hygiene:
  - 1 - 2 times weekly with 1 - 2 sprays in the external ear canal.
  - If necessary (e.g. itching due to dryness), Otocura® can also be used more often without hesitation.
- For preventive purposes:
  - Add 1 - 2 sprays before and, if necessary, after water sports (swimming, diving, etc.) into the external ear canal.

**Notes:**

The bottle is filled generously, because for technical reasons it cannot be completely emptied. It is guaranteed that the indicated quantity can be used.

When using for the first time, you may have to pump 3 or 4 times to fill the tube.

- Do not use if your eardrum is perforated.
- Remove hearing aid, headphone, headset etc. before use and do not insert or put on directly after the use of Otocura® in order to avoid contamination and/or potential damage to the device.
- Do not push the sprayhead into the ear canal.
- After application, excess oil can be removed with a clean cloth.
- The sprayhead can be removed (e.g. for cleaning) and reapplied again.
- Do not use at the same time as other preparations for use on the ear (e.g. ear drops), in case of doubt consult medical service or pharmacist.
- Do not use if there is a hypersensitivity to any of the ingredients.
- Use of cotton buds and other utensils are not recommended for cleaning the ear canal.

**Shelf-life and storage:**

Store in a cool and dry place.

Keep out of reach of children.

Can be used up to 6 months after first use.

Do not use after expiry date.

Date of last revision: 2020-07